


MULTIPLE BAND LIGATOR SET

Manufactured with the best Quality Medical Grade Materials

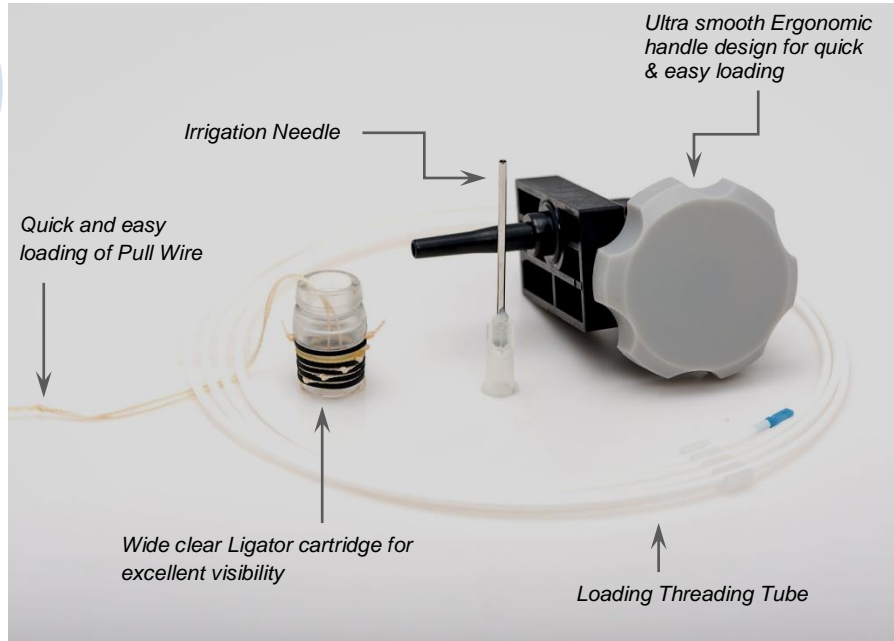

Multi Shot Elastomer Bands with excellent retention.

Natural colored band to alert the operator when only one band remains upto 7 bands deployment

Note: Ligator Cartridge assembly also available without Handle Unit and Irrigation Needle

VARIANTS:

CODE NO	NO OF SHOOTER	FOR ENDOSCOPIC DIAMETER IN MM	CODE NO	NO OF SHOOTER	FOR ENDOSCOPIC DIAMETER IN MM
GMBLS-7-8692	7	8.6 ~ 9.2mm	GMBLS-6-8692	6	8.6 ~ 9.2mm
GMBLS-7-9513	7	9.5 ~ 13mm	GMBLS-6-9513	6	9.5 ~ 13mm
GMBLS-7-1114	7	11 ~ 14 mm	GMBLS-6-1114	6	11 ~ 14 mm
GMBLS-7-8692OV	7	8.6 ~ 9.2mm	GMBLS-6-8692OV	6	8.6 ~ 9.2mm
GMBLS-7-9513OV	7	9.5 ~ 13 mm	GMBLS-6-9513OV	6	9.5 ~ 13 mm
GMBLS-7-1114OV	7	11 ~ 14 mm	GMBLS-6-1114OV	6	11 ~ 14 mm
GMBLS-7-8.5-P	7	8.5 mm	GMBLS-6-8.5-P	6	8.5 mm
GMBLS-7-9511	7	9.5 ~ 13mm	GMBLS-6-9511	6	9.5 ~ 13mm

DEVICE DESCRIPTION :

The Glowtronics Multiple Band Ligator is a variceal ligation system used for endoscopic ligation of esophageal varices, at or above the gastresophageal junction. It consists of an applicator unit, a fixation strap and the ligation bands that are mounted on the barrel.

INTENDED USE :

Used for endoscopic ligation of esophageal varices.

WARNING :

A thorough understanding of the technical principles, clinical applications and risks associated with endoscopic variceal band ligation is necessary before using this device. It is recommended to confirm diagnosis requiring treatment of esophageal varices before set up of Glowtronics Ligator.

CAUTION :

Do not use if the package is damaged, or any components are broken or damaged.
 This device should only be used by or under supervision of physicians trained for this process.
 Any use of this device, other than indicated above, is not recommended.
 Ligator Cartridge, Handle Unit, Pulling Catheter, Irrigation Needle etc. are all supplied clean and sterile.
 Disposal of this device must be as per the standard institutional guidelines and norms for biohazardous medical waste, which are in conformity with state/national laws and regulations in force.
 This contains Latex – Natural rubber which may cause allergic to few individuals.



Glowtronics Pvt Ltd

1D, Hootagalli Industrial Area,

Mysore – 570 018, India Tel: +91-821-240-2461/2462

Email: marketing@glowtronics.com Website: www.glowtronics.com

