


CHEMO PERFUSION SET

Manufactured
with the best
Quality Medical
Grade Materials

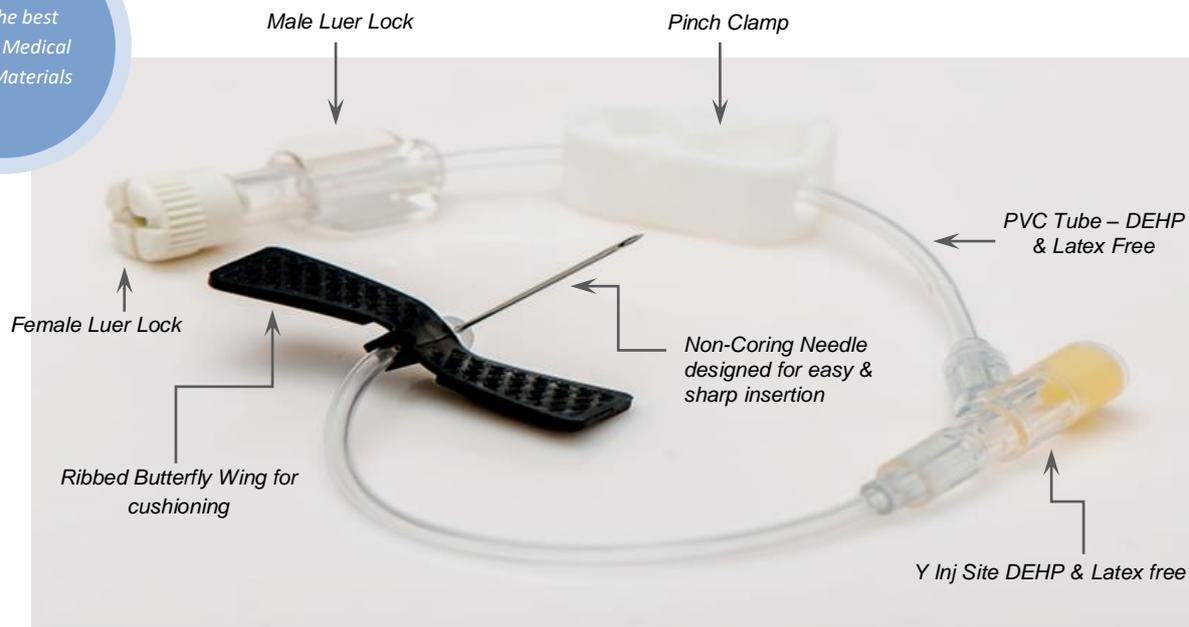

VARIANTS :

CODE NO	GAUGE (G)	LENGTH (L)	ISO 6009 COLOR DESIGNATION
Chemo Perfusion Set 2220	22	20	Black
Chemo Perfusion Set 2225	22	25	Black
Chemo Perfusion Set 2020	20	20	Yellow
Chemo Perfusion Set 2025	20	25	Yellow

DEVICE DESCRIPTION :

The Chemo Perfusion Set is composed of a standard non-coring Huber type needle and administration set consisting of Butterfly wing, Y Injection Site, Pinch Clamp, Male/Female Luer Lock connector and Phthalate free PVC tube. The Needle is bent at a 90° angle and available in standard gauges and lengths.

INTENDED USE :

The device is designed for use with vascular access perfusion system and is intended for use as an intravascular administration set to access surgically implanted subcutaneous vascular ports in a standard manner for the purposes of fluid or drug infusion and blood sampling. Chemo Perfusion Sets are also used in Dialysis, Blood transfusions and Intravenous cancer treatments.

WARNING :

This device should only be used by staff under the supervision of qualified Oncologist thoroughly trained in Chemotherapy procedures. A complete understanding of the technical principles, clinical applications and risks associated in Chemotherapy procedures is necessary before using this device. Ensure sterilization of the device before use.

CAUTION :

These devices are for Single - use ONLY.
Check expiry date prior to use. Do not use after expiry date.
Do not Reuse, Re - process or Re-sterilize.
Do not use if package is opened or damaged when received.
This device must be stored in dry & cool conditions, away from extreme temperature.
Use of this device for any other purpose other than the stated intended use is neither advised nor recommended.



Glowtronics Pvt Ltd
1D, Hootagalli Industrial Area,

Mysore – 570 018, India Tel: +91-821-240-2461/2462

Email: marketing@glowtronics.com Website: www.glowtronics.com

