

DISPOSABLE LAP ELECTRODE L-HOOK WITH CABLE AND CONNECTOR



8mm Bovie Connector



4mm Banana Connector



Manufactured with the best Quality Medical Grade Materials

4mm TPR Cable



Light weight Ergonomic Molded Handle for safe & easy operations

Available with monopolar cable, High Quality Insulation Tubing all the way to the electrode. Ensuring insulation down to the blades (Dielectric Strength - 20 Kv/mm) provides greater patient safety

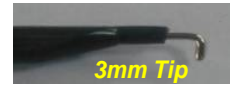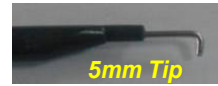

VARIANTS:

CODE NO.	OUTER DIA	WORKING LENGTH	EXPOSED TIP LENGTH	HANDLE COLOR	CONNECTOR USED
M-L-5333F-B-8BV3C	5 mm	335 mm	3 mm	Blue	8mm Bovie
M-L-5333F-B-4BP3C	5 mm	335 mm	3 mm	Blue	4mm Banana
M-L-5335F-B-8BV3C	5 mm	335 mm	5 mm	Blue	8mm Bovie
M-L-5335F-B-4BP3C	5 mm	335 mm	5 mm	Blue	4mm Banana

DEVICE DESCRIPTION :

Glowtronics monopolar disposable laparoscopic electrodes are single use electro-surgical devices. These devices are Unsterilized, unless specifically requested by the customer for EtO Sterilization. Individually packaged in a tyvek pouch.

INTENDED USE :

The disposable laparoscopic electrodes are intended for use in minimally invasive laparoscopic surgical procedures for Cut, Dissect, specimen retrieval, coagulating, cauterizing and other manipulation of tissues and vessels during laparoscopic procedures.

WARNING :

This device should only be used by or under supervision of qualified endoscopists thoroughly trained in Endoscopic procedures. A complete understanding of the technical principles, clinical applications and risks associated in endoscopic procedures is necessary before using this device. Ensure sterilization of the device before use.

CAUTION :

These devices are for Single - use ONLY.
 Check expiry date prior to use. Do not use after expiry date.
 Do not Reuse, Re - process or Re-sterilize.
 Do not use if package is opened or damaged when received.
 This device must be stored in dry & cool conditions, away from extreme temperature.
 Use of this device for any other purpose other than the stated intended use is neither advised nor recommended.



Glowtronics Pvt Ltd
 1D, Hootagalli Industrial Area,
 Mysore – 570 018, India Tel: +91-821-240-2461/2462
 Email: marketing@glowtronics.com Website: www.glowtronics.com



intertek



intertek

